


Atlantis® screw torque recommendations for Atlantis® abutments

The torque recommendations are a reference for your convenience. They might change without further notice in order to follow manufacture's recommendations.

Manufacturer	Implant	Torque Ncm	Screwdriver	
BioHorizons	BIOHORIZONS			
	Internal 3.5, 4.5, 5.7	30	0.050" (1.25 mm) Hex screwdriver	
	Tapered Internal 3.5, 4.5, 5.7	30		
	External implant 3.5	30		
	External implant 4.0, 5.0	30		
Biomet 3i	BIOMET 3i			
	Certain™ 3.25, 4.0, 4/3, 5.0, 5/4, 6.0	20	0.048" Hex screwdriver	
	Certain™ Prevail™ 3/4/3 (3.4), 4/5/4, 5/6/5, 4/3, 5/4, 6/5	20		
	Certain™ XP 4/5, 5/6	20		
	Miniplant™ 3.25	20		
	Osseotite™ 3.75, 4.0, 5.0, 6.0	20		
	Osseotite™ XP 4/5, 5/6	20		
Camlog	CAMLOG			
	CONELOG 3.3	20	1.28 mm Hex screwdriver	
	CONELOG 3.8, 4.3	30		
	CONELOG 5.0	30		
	Screw-line Implant 3.3	20		
	Screw-line / Root-line Implant 3.8, 4.3, 5.0, 6.0	20		
	Dentsply Sirona	DENTSPLY SIRONA		
Ankylos® C/X - C/ 3.5, 4.5, 5.5, 7.0 (non-indexed)		15		1.0 mm Hex screwdriver
Ankylos® C/X - /X 3.5, 4.5, 5.5, 7.0 (indexed)		15		
OsseoSpeed® EV 3.0 - Green		25	0.050" Hex screwdriver	
OsseoSpeed® EV 3.6 - Purple		25		
OsseoSpeed® EV 4.2 - Yellow		25		
OsseoSpeed® EV 4.8 - Blue		25		
OsseoSpeed® EV 5.4 - Brown		25		
OsseoSpeed® Profile EV 4.2 Yellow		25		
OsseoSpeed® Profile EV 4.8 Blue		25		
OsseoSpeed® / OsseoSpeed® TX 3.0 - Yellow		15		
OsseoSpeed® / OsseoSpeed® TX 3.5, 4.0 - Aqua		20		
OsseoSpeed® / OsseoSpeed® TX 4.5, 5.0 - Lilac		25		
OsseoSpeed® 4.0 - Blue (Previous)		25		
OsseoSpeed® TX Profile 4.5, 5.0		25		
Xive® S 3.0		24	1.22 mm Hex screwdriver	
Xive® S 3.4, 3.8, 4.5, 5.5		24		
Frialit™ 3.4, 3.8, 4.5, 5.5		24		
Keystone Dental	KEYSTONE DENTAL			
	PrimaConnex™ SD 3.3, 3.5, Genesis™ 3.8	30	Keystone Quad Driver	
	PrimaConnex™ RD 4.0, 4.1, Genesis™ 4.5	30		
	PrimaConnex™ WD 5.0, Genesis™ 5.5, 6.5	30		

Please turn over 

Atlantis[®] screw torque recommendations for Atlantis[®] abutments

Manufacturer	Implant	Torque Ncm	Screwdriver
Nobel Biocare	NOBEL BIOCARE		
	NobelActive™ 3.0	15	Nobel Biocare Unigrip
	NobelActive™/NobelReplace™ Conical Connection NP 3.5	35	
	NobelActive™/NobelReplace™ Conical Connection RP 4.3, 5.0	35	
	NobelReplace™ NP 3.5	35	
	NobelReplace™ RP 4.3	35	
	NobelReplace™ WP 5.0	35	
	NobelReplace™ 6.0	35	
	Brånemark System™ NP 3.3	35	
	Brånemark System™ RP 3.75, 4.0	35	
Brånemark System™ WP 5.0, 5.5	35		
Osstem	OSSTEM		
	Osstem TS Mini 3.5	20	1.2 mm Hex screwdriver
	Osstem TS Regular 4.0, 4.5, 5.0, 6.0, 7.0	30	
Straumann	STRAUMANN		
	Bone Level 3.3 NC (Narrow CrossFit™)	35	SCS screwdriver
	Bone Level 4.1, 4.8 RC (Regular CrossFit™)	35	
	Standard/Standard Plus 4.8 RN (Regular Neck)	35	
	Standard/Standard Plus 4.8 WN (Wide Neck)	35	
Standard Plus 3.3 NN (Narrow Neck)	35		
Zimmer Dental	ZIMMER DENTAL		
	Tapered Screw-Vent 3.5, 4.5, 5.7/ Tapered Screw-Vent 3.5, 4.5, 5.7 friction fit	30	1.25 mm Hex screwdriver
	Screw-Vent 3.3, 3.7, 4.5/ Screw-Vent 3.3, 3.7, 4.5 friction fit	30	
	SwissPlus™ 4.8	35	

All trademarks and company names are the property of their respective owners.

Medical and legal disclaimer

The information (including torque recommendations) is provided for information purposes only and does not constitute advice for medical or treatment purposes, nor a legal contract, between Dentsply Sirona Implants and any person or entity. Although all reasonable effort is made to present accurate information (including torque recommendations), Dentsply Sirona Implants makes no guarantees of any kind, including but not limited to that it may be outdated, invalid or subject to debate. The information is provided "as is" without warranty of any kind, either express or implied, including without limitation, the implied warranties of merchantability, fitness for use of a particular purpose, or non-infringement.

You agree that neither Dentsply Sirona Implants, nor any other company within the Dentsply Sirona international group, will be liable to you for any loss or damages, either actual or consequential, arising out of or relating to these terms, or to your (or any third party's) reliance upon information (including torque recommendations) obtained from Dentsply Sirona Implants. In particular, Dentsply Sirona Implants will have no liability for any consequential, indirect, punitive, special or incidental damages, whether foreseeable or unforeseeable, arising out of or relating to these terms, or to your (or any third party's) reliance upon information (including torque recommendations) obtained from Dentsply Sirona Implants, whether based in contract, tort, statutory or other law, except only in the case of death or personal injury where and only to the extent that applicable law requires such liability.